

# Saliva Collection Kit

## User Manual

Version: A2.0

Edit Date: 2020-11-18

### 【Product Name】

Saliva Collection Kit

### 【Model Number】

SCK-TA, SCK-TB, SCK-TC, SCK-TD, SCK-TE, SCK-TF

SCKPL-TA, SCKPL-TB, SCKPL-TC, SCKPL-TD, SCKPL-TE, SCKPL-TF

### 【Application】

For the collection of human saliva samples.

This product collects saliva samples secreted from the oral cavity, and mixes the collected saliva with the saliva preservation liquid evenly, ensuring the integrity of the DNA in the saliva sample and its long-term storage at room temperature.

### 【Composition】

Composed of funnel, cap, tube and/or preservation solution.

### 【Storage conditions and validity】

Storage conditions: normal atmospheric temperature

Validity: 2years


### 【Instructions】



Clean the mouth



Spit saliva lightly

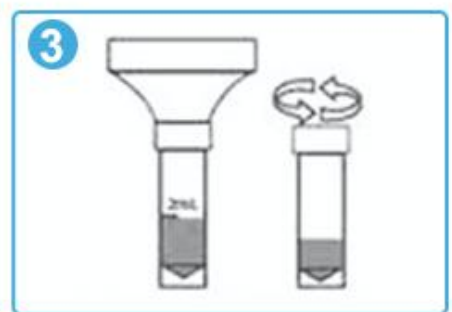


Unscrew the cap



Pour into the funnel



Unscrew the collection tube

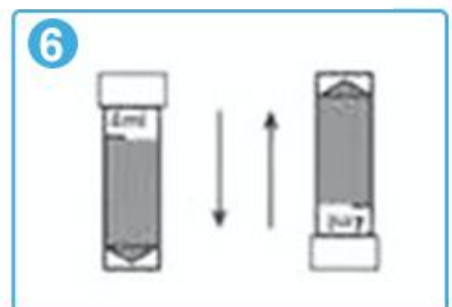

Mix well



30 minutes before collecting a saliva sample, please wash away the debris in the oral cavity with absorption.



Press the tongue against the root of the upper or lower jaw to enrich the saliva, and gently spit the saliva into the collecting funnel until the liquid saliva (non-bubble) reaches the height of the 2m scale line.

#### **【Precautions】**

1. Before use, please check the sealing integrity of the package. Do not use damaged packaging.
2. This product is for one-time use.
3. Treat as medical waste after use.

#### **【Warnings】**

1. This product is a one-time product, it is strictly prohibited for multiple people to use or reuse.
2. During use, if there are cracks in the appearance of the product, it is forbidden to use it.
3. If the product fails, please contact us or your local dealer.

#### **【Manufacturer information】**

Manufacturer: Shenzhen J.able Bio Co.,Ltd.

Address: 401,13th Building, Lihao Industrial Park, Ainan Road #78, Longgang District, 518116, Shenzhen, Guangdong Province, China

Tel: 0755-28591602

E-mail: [info@jablebio.com](mailto:info@jablebio.com)

#### **【European Authorized Representative】**

Name: Luxus Lebenswelt GmbH

Add: Kochstr.1, 47877, Willich, Germany

Tel: 0049- 1715605732

E-mail: [info.m@luxuslw.de](mailto:info.m@luxuslw.de)